

Easy to Load.



1. Place cartridge, luer tip first, into open end of holder. Twist Blue lock to close.

Easy to Unload.



1. To disengage, turn plunger rod counter clockwise. Pull back plunger fully. Twist blue lock open to release.



2. To engage, turn plunger rod clockwise. Proceed with injection in normal manner, leaving tip guard on until just before use.



2. Place thumb on cartridge and slide back. Invert holder and release used cartridge unit into disposal bin.
Do not recap!
Holder autoclavable if sterility is desired.