

CLAVE[®] Connector

PRODUCT COMPATIBILITY NOTICE

ICU Medical, Inc. is aware that the CLAVE Connector is not compatible with every device utilizing a male luer. Small internal diameters of certain male luer devices may cause damage to the CLAVE resulting in leakage, reduction or loss of fluid flow. DO NOT use any male luer device unless specified here, or written verification of compatibility has been received from ICU Medical, Inc. Please call (800) 824-7890 for verification.

****The CLAVE Connector Contains no Natural Rubber Latex****

The CLAVE Connector is known to be compatible with the following devices utilizing male luers:
STANDBORE ADMINISTRATION, PUMP AND EXTENSION SETS

Hospira	Adv Medical	All Medical	Baxter	BD
Block	B. Braun	Churchill	Codan Medlon	3M
IMED/Alaris	IVAC/Alaris	Medex	MPS	Secure
Solopak	Medical Specialties	Sabratek	Sims/Deltec	I-Flow
BMP	Setfinder	Setsource		

MINIBORE EXTENSION SETS

Hospira	Adv Medical	All Medical	Baxter	Churchill
IMED/Alaris	IVAC/Alaris	Medex	BMP	Solopak
3M	Codan Medlon	Setfinder	B. Braun	

MICROBORE EXTENSION SETS

Hospira	Adv Medical	Baxter	Churchill	MPS
IMED/Alaris	B. Braun	Triad	Valumed	
Medical Specialties		BMP	Setfinder	

ULTRA-MICROBORE EXTENSION SETS

Hospira	Baxter	MPS	BMP	Setfinder
Medical Specialties				

SYRINGES AND PRE-FILLED SYRINGES

Hospira Ansy	Medrad	Monoject	JMS
Terumo (except 1cc Tuberculin)	ROCAP	Solopak	BD
IMS (with ICU No. CS25)	IMS Luer-Jet	Abboject	Nipro
Carpject (Luer Tip)			

STOPCOCKS

B. Braun (except # D-600)	Hospira # 3233	Medex # MX431-1L	BMP
---------------------------	----------------	------------------	-----

ICU Medical, Inc., 951 Calle Amanecer, San Clemente, CA. 92673

<http://www.icumed.com>

CAUTION: Federal (USA) law restricts this device to sale by or on the order of a physician.

CLAVE is a registered trademark of ICU Medical, Inc., San Clemente, CA

US Patents: 5,199,947; 5,344,414; 5,685,866; 5,688,254; 5,690,612; 5,694,686; 5,695,466; 5,700,248; 5,738,663; 5,776,116; 5,873,862; 5,901,942; 5,928,204